

Quick reference guide for Lifeline Digital (022-25-9xx)



Yellow Extra button



Green Cancel button



Red Alarm button

Enable programming mode:

- a) Press and hold the green Cancel button, then press and hold the red Alarm button and the yellow Extra button.



- b) Release all buttons when the device emits a rising sound signal and announces "Programming mode",



The LED indicators start chaser sequence and the button LEDs flash in unison.



- c) You can now:

- [Adjust speaker volume, page 1](#)
- [Adjust LED, page 1](#)
- [Activate Radio test mode, page 1](#)
- [Check cellular signal, page 2](#)
- [Connect a peripheral, page 2](#)
- [Disconnect a peripheral, page 2](#)

Adjust speaker volume:

- a) Press and hold the yellow Extra button.



- b) When the device announces "One", release the button.



- c) Adjust speaker volume:

- Press the red Alarm button to increase volume.



- Press the yellow Extra button to decrease volume.



- d) Press the green cancel button to save settings and exit.



Adjust LED intensity:

- a) Press and hold the yellow Extra button.



- b) When the device announces "Two", release the button.



- c) Adjust LED intensity :

- Press the red Alarm button to increase LED intensity.



- Press the yellow Extra button to decrease LED intensity.



- d) Press the green cancel button to save settings and exit.



Radio test:

- a) Press and hold the yellow Extra button.



- b) When the device announces "Three", release the button.



- c) When the device announces "Radio test mode Activate transmitter now", activate/trigger the peripheral you want to test:



- Connected and battery is OK.
- Not connected and battery is OK.
- Connected but battery is low.
- Not connected and battery is low.

- d) Press the green cancel button to exit.



Check cellular signal:

- a) Press and hold the yellow Extra button.



- b) When the device announces "Four", release the button.



The device announces the current cellular signal strength on a scale of 1 to 5:

- "Cellular signal strength is One" for poor signal.
- "Cellular signal strength is Five" for excellent signal.



- c) Press the green cancel button to exit.



Connect a peripheral:

- a) Press and hold the red Alarm button.



- b) When the device announces "Auto pairing mode", you can either:



- Release the button to enable auto pairing mode.
In auto pairing the peripheral is stored in the first available radio sensor position.
- Continue to hold the button until the device announces the number of the radio sensor position you want to select, then release the button.

- c) When the device announces "Activate transmitter now", activate/trigger the peripheral.



- d) Press the green cancel button to save settings and exit.



Disconnect a peripheral:

- a) Press and hold the red Alarm button.



- b) When the device announces "Auto pairing mode", you can either:



- Release the button to enable auto pairing mode.
- Continue to hold the button until the device announces the number of the radio sensor position you want to select, then release the button.

- c) Press the yellow extra button.



- If you have selected auto pairing mode, the device announces: "Erasing. Activate transmitter now".



- If you have selected a specific radio position, the device announces "Erasing" and disconnects the peripheral.

- d) Press the green cancel button to save settings and exit.

