

Cleaning Guide

This guide incorporates all individual cleaning instructions for the telecare-based product range as given in user documentation. The texts are extracts from the manufacturers' guidelines.

PRODUCT NAME	PART NUMBER	CLEANING INSTRUCTIONS
Alert-it Companion Epilepsy Monitor	S9001001	It is recommended to regularly clean both units by wiping with cotton wool pads moistened (compressed until dripping stops) with a mild detergent (0.5% washing-up liquid) solution or by using an alcohol or baby wipe. Avoid getting any liquid into containers.
Ambient Temperature Sensor	61005/40	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Bed Occupancy Mat (under mattress)	D4106009A	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Carbon Monoxide Alarm	68005/98	Clean the outside case by occasionally wiping with a clean damp cloth. Do not use any cleaning agents, bleaches, detergents or polishes, including those in aerosol cans. Avoid spraying air fresheners, hair spray, paint or other aerosols near the CO Alarm. Do not place air fresheners near the CO Alarm.
CareAssist	68100/01	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Chair Occupancy Sensor Mat	D4106010A	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Flood Detector	67005/37	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Heat Detector	68005/71	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
iVi Intelligent Pendant	P68005/45	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Lifeline GSM	56000/320	Dust the home unit with a soft cloth which can be moistened with a gentle detergent if required. Ensure that no moisture goes through the speaker grill.
Lifeline Vi/Vi+	57000/320 - Lifeline Vi 57100/320 - Lifeline Vi+	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Lifeline Smart Hub	57200/810	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Minuet Watch	67005/21	Use only clean water and soap.

PRODUCT NAME	PART NUMBER	CLEANING INSTRUCTIONS
Motion Sensor	61005/35	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
MyAmie	P68005/10	Use only clean water and soap.
Pivotell Advance Medication Dispenser	67005/85	Dispenser – wipe with a damp cloth. Internal tray – may be washed in warm water or in a dishwasher (max. 70°C)
Pressure Mat	GS140	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Smoke Alarm	68005/70	Clean your alarm regularly. Use the flat open end of your vacuum cleaner to remove dust and cobwebs from the side slots where the smoke enters. To clean the cover, wipe with a damp cloth and dry thoroughly.
Universal Sensor	61005/30	Clean using an IPA (Isopropanol) wipe. Do not use IPA spray.
Vibby Fall Detector	67605/22	The manufacturer of the Vibby recommends it is washed once a week to stop any dirt or dust accumulating that could cause irritation. The Vibby may be cleaned with a damp cloth and mild detergent. Remember that the Vibby will not detect falls while it is not being worn therefore you should put the Vibby back on again as soon as you have finished cleaning it.